

Because *Septofil* is a natural fibre tubular knitted cord with interlaced stitch network.

1 / Easy positioning

The unique manufacturing process ensures the cord does not fray.

It tends to stay in place until removal.

2 / Maximum impregnation

Septofil's hollow tubular structure and the absence of synthetic fibers, which allow the cord to swell when impregnated, provide Septofil with an exceptionally high absorption capacity.

Reference	Diameter	Colour
Thick	1.0 mm	green
Medium	0.8 mm	lilac
Thin	0.65 mm	orange
Ultra-thin	0.5 mm	midnight blue

Other products from Septodont which will make your life easier

Racestyptine Solution

A non epinephrine based gingival retraction solution

Endosolv

For removing canal fillings

Proviscell

Eugenol-free temporary cement



Septodont of Canada, Inc.

Distributor of Septodont and Novocol Products
Cambridge, ON 800 647 0643
www.septodont.ca

*Why choose . .
. . Septofil?*




Septofil

Non-impregnated tubular cord
for gingival retraction



A natural approach to gingival retraction

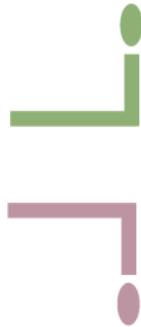

Directions for use

Septofil produces a temporary mechanical retraction of the sulcus, which can be aided by the additional use of Racestypine Solution, prior to taking an impression or carrying out other procedures.

If gingival retraction is required in more than one tooth, treat each tooth individually.

Pull the cord out of the bottle and cut the required length: the perimeter of a tooth at the crown can vary from 2 to 7 cm. Therefore, a sufficient length of cord should be cut so that when the cord is wound around the tooth, the ends overlap. This makes the removal of the cord easier for the practitioner, particularly after impregnation.

1/ Place the cord in the gingival crevice using a blunt metallic instrument to avoid injuring epithelial attachment. Start with the interproximal space as this is where the sulcus is usually deepest. Then, proceed clockwise until you have encircled the tooth.

2/ Follow directions of use indicated for the retraction solution or gel.

3/ The cord should be removed immediately prior to taking an impression, except when the double impression technique is used. To ease removal, slightly moisten the cord.



Septofil's knitted design tends to stay in place



1



2



3

4 diameters / 4 colours

- Thick - Green
- Medium - Lilac
- Thin - Orange
- Ultra-thin - Midnight blue

These colours are easy to identify in the sulcus.