COMPOSITION

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PROPERTIES

TRAY-NET softens alginates, or plaster on impression trays by simple immersion followed by rinsing in water, without scrubbing.

TRAY-NET can be used on all plastic or metallic impression trays (except aluminium trays) and also to clean spatulas and alginate mixing bowls.

INDICATIONS

Cleaning powder for impression trays

INSTRUCTIONS FOR USE

- Using the spoon, put 5 measures of powder into an adapted container (tank) per liter of water.

- Fill the tank with hot water.
- Dip the impression trays into the TRAY-NET solution
- Let stand into the solution for a few hours or, preferably, overnight and then rinse the impression trays under running water
- If necessary, a slight scrubbing can be performed to eliminate the remaining alginates

NOTES

For maximum efficiency the solution should be changed after each use
A kilo of powder allows to reconstitute approximately 8 liters of solution at most
TRAY-NET has no effect on silicone or other elastomeric impression materials.

WARNINGS AND PRECAUTIONS FOR USE

Close correctly the bag after use
Avoid leaving the spoon in contact with the powder during storage (out of the bag)
Refer to safety data sheet – available upon request

STORAGE

Protect from moisture

PRESENTATION

Box containing 1 kg of TRAY-NET powder with spoon.

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