# Cimpat

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## Indications

Paste for temporary filling

- **adheres to the dentine,**
- **hardens rapidly in the cavity,**
- **expands slightly in the process of setting,**
- **ensures a hermetic seal.**
- CIMPAT PINK can be used over cotton wool dressings, or directly at the base of the cavity. It is equally an excellent filling material for use over arsenical dressings. CIMPAT PINK can be used to set temporary crowns.
- CIMPAT WHITE, more plastic than CIMPAT PINK, is generally designed for dressings that are likely to cause pain on compression. CIMPAT WHITE **may be removed with a probe.** Because of its friable consistency, CIMPAT WHITE should only be used for short time fillings.
- CIMPAT N is the thickest of the four CIMPAT. It is designed for longer lasting temporary filling.

## Precautions for use

Like any filling materials, CIMPAT remains in place only when the cavity is properly trimmed.

CIMPAT is softened during setting by certain solvents contained in liquid or pastes used for canal treatment; Therefore after any canal treatment and before filling with CIMPAT it is required to clean the cavity thoroughly and allow to dry.

## Important

For easy handling, use a warm spatula.

## Presentations

CIMPAT PINK, WHITE and N :  
Jar containing 25 g.  
Box containing 6 jars of 25 g.

*For professional use only in the practice of dentistry.*