

Cimpat S

Obturation provisoire prête à l'emploi

Formule

Oxyde de zinc 41,29 %, sulfate de calcium 28,00 %, sulfate de zinc 13,20 %, excipients q.s.p. 100 %.

Indications

Pâte pour obturations provisoires

- adhère à la dentine,
- durcit rapidement dans la cavité,
- se dilate légèrement en cours de prise,
- assure une obturation hermétique.

CIMPAT S peut servir pour la restauration temporaire des cavités entre la préparation de la dent et la fixation définitive des inlays ainsi que pour la fixation des couronnes provisoires.

Précautions d'emploi

CIMPAT, comme toutes les obturations, ne reste en place que dans une cavité convenablement taillée.

CIMPAT est ramolli en cours de prise par certains solvants contenus dans les liquides ou pâtes de traitement canalair. Il y aura lieu, après un traitement canalair et avant obturation par CIMPAT, de nettoyer les parois de la cavité, puis de laisser sécher.

Important

Pour une manipulation facile, utiliser une spatule tiède.

Présentation

Boîte contenant 4 seringues de 7,5 g.

Réservé à l'usage professionnel en art dentaire.

Distribué par:
NOVOCOL PHARMACEUTICAL OF CANADA INC.
CAMBRIDGE - ONTARIO N1R 5S9

Cimpat S

Temporary filling ready to use

Formula

Zinc oxide 41.29 %, calcium sulphate 28.00 %, zinc sulphate 13.20 %, excipients q.s.ad. 100 %

Indications

Paste for temporary filling

- adheres to the dentine,
- hardens rapidly in the cavity,
- expands slightly in the process of setting,
- ensures a hermetic seal.

CIMPAT S can be used as for the temporary restoration of cavities in the intervals between tooth preparation and the permanent cementation of completed inlays and as a cementing medium for all forms of temporary crowns.

Precautions for use

Like any filling materials, CIMPAT remains in place only when the cavity is properly trimmed.

CIMPAT is softened during setting by certain solvents contained in liquid or pastes used for canal treatment; Therefore after any canal treatment and before filling with CIMPAT it is required to clean the cavity thoroughly and allow to dry.

Important

For easy handling, use a warm spatula.

Presentation

Box containing 4 syringes of 7.5 g.

For professional use only in the practice of dentistry.

Distributed by:
NOVOCOL PHARMACEUTICAL OF CANADA INC.
CAMBRIDGE - ONTARIO N1R 5S9

