# Endosolv

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# Endosolv

## Product for removing canal fillings

### Composition

ENDOSOLV E (for eugenates)	
Tetrachloroethylene .....	92,30 g
Excipients qs ad .....	100,00 g
ENDOSOLV R (for phenolic resins)	
Formamid .....	66,50 g
Phenylethyl alcohol .....	33,50 g

### Properties

ENDOSOLV E allows the softening of temporary fillings of the normal zinc oxide-eugenol variety, while ENDOSOLV R aids in the softening of phenolic resin type fillings.

ENDOSOLV does not provoke excessive salivation if it comes into contact with the mucosa.

### Method of use

#### Recommended one visit method :

- 1° Clean out, by mechanical means, the pulp-chamber and entrance to the canals.
- 2° Place a drop of ENDOSOLV into the chamber and clear the canal, using first a probe, then a broach, dipping the point of the instruments in ENDOSOLV before each application. The instrument must be removed from the canal as soon as a certain resistance is felt, then wipe and repeat the operation a number of times, until reaching the apex.

#### Recommended two visits method :

To prepare for the removal of the canal filling at a subsequent visit :

- 1° Clear out the pulp-chamber mechanically, and if possible, the entrance to the canal.
- 2° Place into the cavity a small cotton pellet soaked in ENDOSOLV. Compress slightly.
- 3° Fill with a sealing cement.  
Re-open either the next day or some days later.  
The paste will have desintegrated and the mechanical reaming will present no difficulty.

### Special warnings

A tooth treated with ENDOSOLV must be carefully dried before a new restoration made of eugenol resine or acrylic, is placed.

ENDOSOLV E :

Xn - Harmful, contains tetrachloroethylene.

N - Dangerous for the environment.

Possible risks of irreversibles effects.

Toxic to aquatic organisms.

May cause long-term adverse effects in the aquatic environment.

ENDOSOLV R :

T - Toxic, contains formamid.

Possible risk of harm to the unborn child.

### Precautions for use

ENDOSOLV E :

Wear suitable protective clothing and gloves;

ENDOSOLV R :

Avoid exposure, obtain special instructions before use.

In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

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Bottles of 13 ml.

*For professional use only in the practice of dentistry.*

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