Isodan

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Isodan

Treatment of dentinal hypersensitivity

Composition

- . Potassium nitrate, HEMA, sodium fluoride
- . Excipients

Properties

ISODAN is used for the treatment of dentinal hypersensitivity caused by all irritating agents of the dentin : physical (cold, contact, heat), chemical (acid erosion), physiological (abrasion, tooth erosion).

Indications

Dentinal hyperaesthesia.

Can be used on all exposed dentin surfaces, crown preparations and cervical erosion.

Can be used under amalgams and any restorative material, in all restorative situation.

Contraindications

Hypersensitivity to one of the components.

Instructions for use

- Isolate one or several teeth to be treated using a cotton roll,
- Carefully dry the surface of the lesion to be treated,
- Pour a drop of product into a small jar,
- Take a small quantity with a microbrush,
- Apply a thin film of the product,
- Carefully spread it with air blast,
- Reapply two or three times until the surface is shiny.

IN CASE OF PERSISTING SYMPTOMS, REAPPLY AT ONE WEEK INTERVAL.

Warning and precautions for use

Xi : irritating to eyes and skin. May cause sensitization by skin contact. Avoid contact of the material with skin or eyes. In case of contact, wash thoroughly with water.

Avoid brushing the treated site for 24 hours.

Avoid drinking or eating any acid food for 24 hours.

Storage

Should be stored at a temperature below 25°C.

Presentation

1 x 5 ml bottle + 20 application brushes.

For professional use only in the practice of dentistry.

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