Proviscell

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Eugenol-free temporary cement

Properties

PROVISCELL, **eugenol-free** temporary cement is simple to apply and can be easily removed from plastic and metal crowns. PROVISCELL has no influence on the hardening of sealing cement base or resin.

Indication

Temporary sealing.

Directions for use

PROVISCELL should be used at room temperature (18 - 23 °C) and at a normal humidity level (45-55 %). Working time is longer in case of cooling glass slab.

Pour equal amounts of base and catalyst into a mixing pad or glass plate.

Mix then the two pastes for approximately 20 seconds until the material takes on a homogeneous consistency. Use a spatula made of plastic or rustproof metal for mixing.

Apply a thin layer of PROVISCELL inside the temporary crown or bridge and fasten on the prepared tooth. PROVISCELL flows freely while the crown or bridge is being put in place.

PROVISCELL remains workable for about 1 1/2 to 2 minutes and sets in the mouth within 4-6 minutes.

3-5 minutes after PROVISCELL has set, remove any excess material.

Remarks

To obtain a softer material and a **longer setting time** add just a little **more of the base paste** or a small trace of vaseline. A moist environment (humid stump tooth) facilitates subsequent temporary cement removal.

A harder and **faster setting material** is obtained by adding **more catalyst paste**.

Special warnings

In case of exposed pulp or thin layer of dentin, it is recommended to first apply a cavity liner. Do not use PROVISCELL if the patient is known to be allergic to fatty acid and colophony.

Storage

Keep PROVISCELL at temperature below 23 °C and close tubes after use.

Presentation

Set containing :

- One 25 g tube of base,
- One 25 g tube of catalyst,
- One mixing pad.

For professional use only in the practice of dentistry.

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