Tray-net

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Tray-net

Cleaning product for Impression trays

Composition

Sodium citrate, cetrinide, excipients.

Properties

TRAY-NET disintegrates alginates, or plaster on impression trays by simple immersion followed by rinsing in water, without scrubbing.

TRAY-NET can be used on all plastic or metallic impression trays (except aluminum trays) and also to clean spatulas and alginate mixing bowls.

TRAY-NET is safe for hands.

Directions for use

- Using the measuring spoon, put five measures of powder into a suitable container.
- Add one liter of hot water.
- Dip the impression trays into the TRAY-NET solution.
- Let stand into the solution for a few hours or, preferably, overnight and then rinse the impression trays under running water (only "nickel silver" trays and plastic trays used with an adhesive require light scrubbing).

Note

- For maximum efficiency the solution should be changed after each use ; it is, however, possible to keep the prepared solution for the treatment of 15 to 20 trays, in groups of 5 or 6. TRAY-NET has no effect on silicone or other elastomeric impression materials.

Presentation

Pot containing 1 kg of TRAY-NET powder with measuring spoon.

For professional use only in the practice of dentistry.